Handicare Stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T +31 (0)72 576 88 88
traplift@handicare.com

Handicare Accessibility Ltd.
82 First Avenue,
Pensnett Estate Kingswinford,
West Midlands DY6 7FJ,
England T +44 (0)1384 408700
enquiries@handicare.co.uk

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Savaria Canada
2 Walker Drive Brampton
ON L6T 5E1 Canada
T 1.905.791.5555
T 1.855.728.2742
info@savaria.com

AA28617_13


















Handicare 1100 User Manual

Manual Style Seat
Powered Style Seat

- EN
- FR
- DE
- IT
- ES
- NL
- CZ
- DK
- SE
- NO
- PT
- CN
- JP
- TR
- AR
- PL

All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual can also be changed without prior warning. This manual is intended for the standard model of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications which differ from those of the standard model of the lift supplied to you. For information concerning adjustment, maintenance or repairs not provided in this manual, you are asked to contact the technical department of your supplier. This manual has been compiled with every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof.

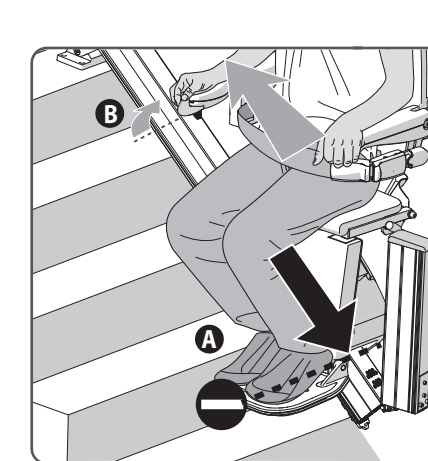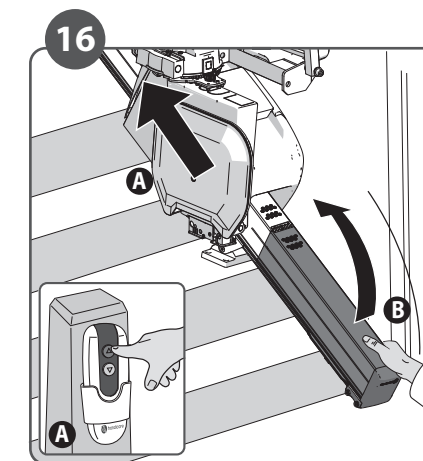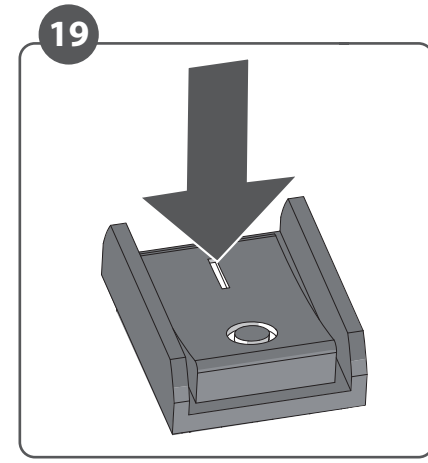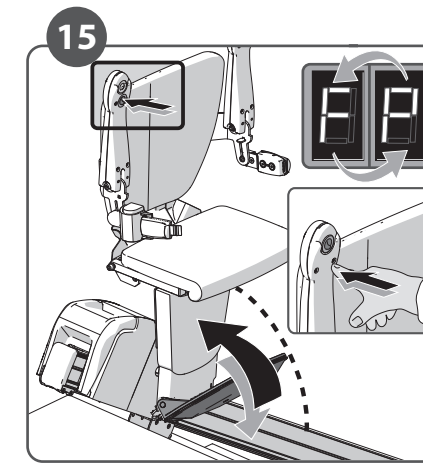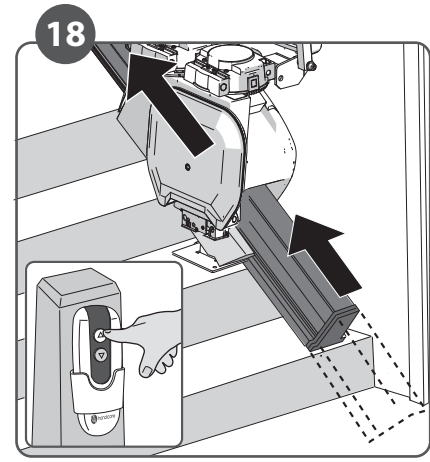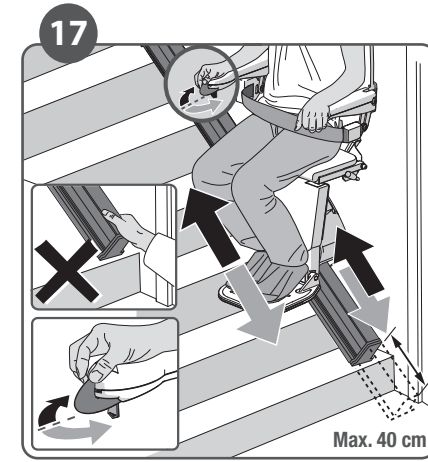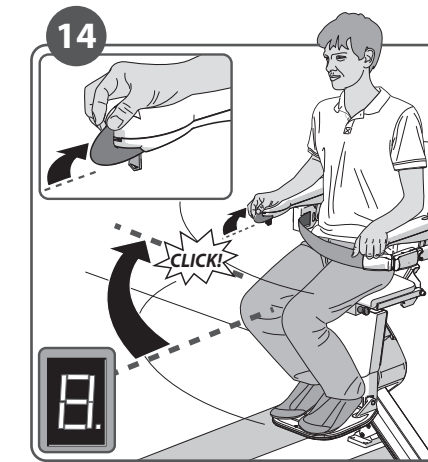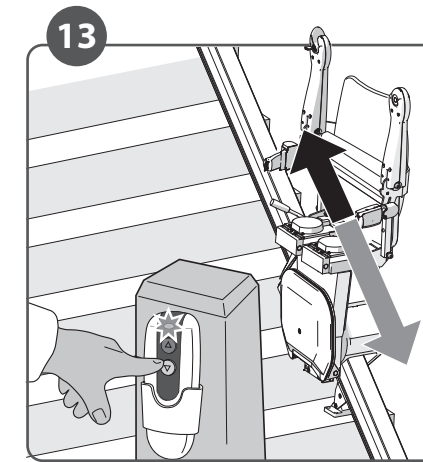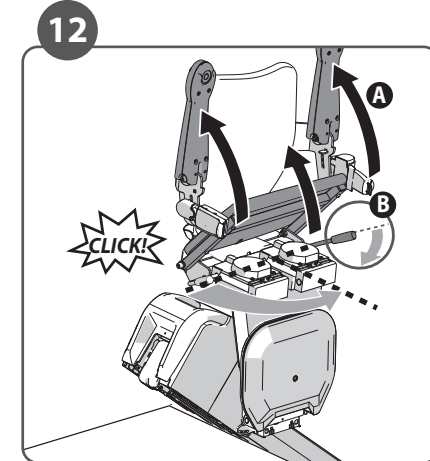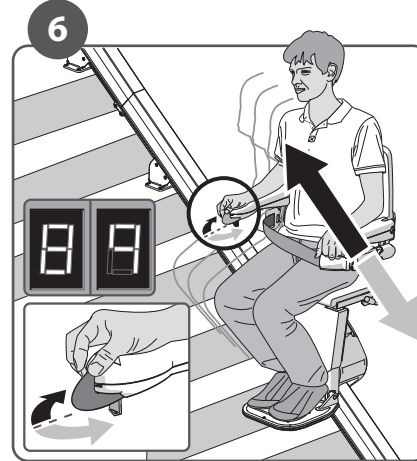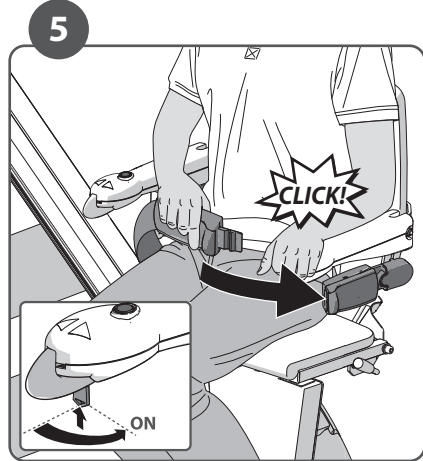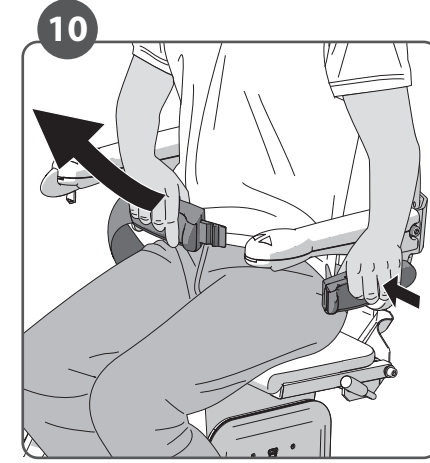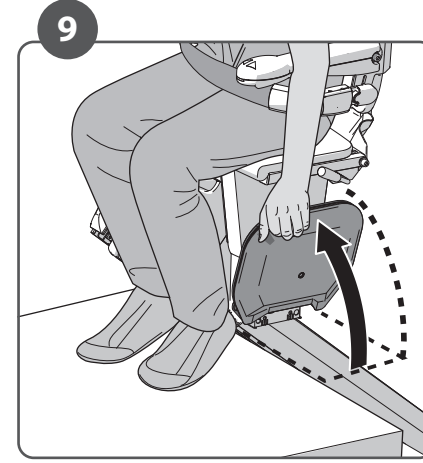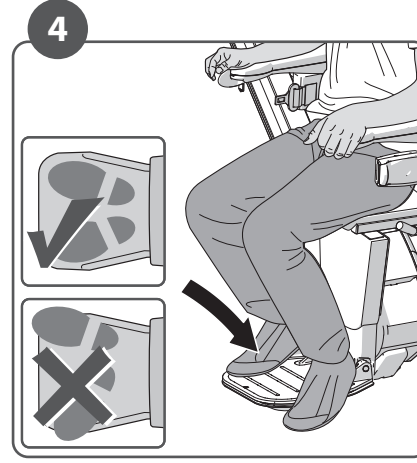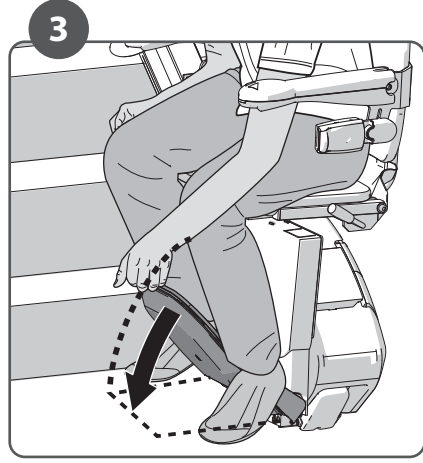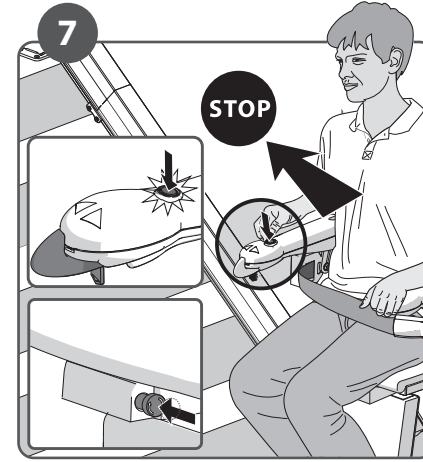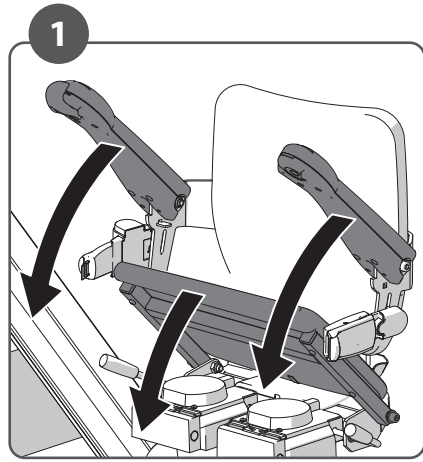ENGLISH	2	EN
FRANÇAIS	6	FR
DEUTSCH	10	DE
ITALIANO	14	IT
ESPAÑOL	18	ES
NEDERLANDS	22	NL
ČEŠTINA	26	CZ
DANSK	30	DK
SVENSKA	34	SE
NORSK	38	NO
PORTUGUÊS	42	PT
中國人	46	CN
ジャパニーズ	50	JP
TÜRK	54	TR
أرأبش	58	AR
POLSKIE	62	PL

- 1 Fold down**
Fold down the armrests and seat.
 - 2 Sit down**
Sit down on the seat.
 - 3 Unfold footrest**
Fold down the footplate.
 - 4 Place feet**
Place your feet on the footplate.
Ensure that both feet are completely on the plate.
 - 5 Safety first**
Fasten the seat belt securely, ensuring that it clicks into the holder. Put the key in the switch and turn it ninety degrees clockwise. Check that the chair is in the locked position.
 - 6 Ride**
Move and hold the joystick in the desired direction of travel. The display shows a "8/9", the lift gradually starts to move.
 - 7 Stop button**
In an emergency, press the stop button once. The lift stops straight away. To release the lift, press the stop button again.
Optional: the emergency stop can be released by turning the emergency button.
 - 8 Swivel**
The lift stops automatically at the end of the rail. You can now swivel the seat manually, making it easier to get on and off. To do this, push down the lever{A}. Swivel the seat until you hear a 'click'. The seat is now in the correct position{B}.
(Optional powered swivel > 14)
 - 9 Fold footrest**
Fold up the footplate.
(Optional powered footrest > 15)
 - 10 Loosen belt**
Unfasten the seat belt.
 - 11 Stand up**
Stand up from the seat.
 - 12 Fold up**
Fold up the armrests and seat {A}. Push the handle, located to the left or right under the seat, downwards {B}. Swivel the seat until you hear a 'click'. The seat is now in the locked position.
 - 13 Park**
Press and hold the button on the remote control until the lift reaches a stop point. Upon arrival, the lift stops automatically.
- ## OPTIONS
- 14 Powered swivel**
The POWERED SWIVEL SEAT option let you swivel the seat automatically to face the direction you are going to walk in. To do this, push and hold the joy-stick in the direction of travel.
 - 15 Powered footrest**
To operate a powered footrest press the switch under the armrest. Always ensure the footplate is in the down position during travel.

16 Hinged rail (manual)

Before raising the manual hinged track{B} park the stairlift above the hinge, or at the top of the stairs{A}. When your stairlift is not in use it should always be positioned away from the hinged section with the hinge in the raised position.

When the lift moves against a folded hinge it will stop automatically{A}. The lift can still ride in the upwards direction{B}.

17 Slide track

If the lift has a sliding track, both the chair and slide track will move at the same time. Once the slide track has reached the end of its travel the lift will continue its travel. Keep the toggle pressed in the same direction. In the case of a powered swivel: If the lift has reached the end of its travel at the top of the stairs and does not automatically swivel, the user should release the toggle and then press the toggle again to swivel the seat.

18 Slide track (park)

When the lift is not in use it should be moved away from the bottom of the stairs so that it does not cause an obstruction. To park the lift, press and hold the UP button on the handset.

19 On/off (option)

In case of power-out in the building, switch the power off from the lift to avoid battery drainage.

Check these points in the event of a malfunction:

1. Is the seat in the correct position before departure?
2. Are seat and armrests down?
3. Is the elevator on? (key switch)
4. Has the emergency stop been activated?
5. Is the seatbelt closed?

In all other cases contact your supplier.

FOREWORD

Your Handicare 1100 stairlift is a sophisticated product for indoor use only that has been specifically developed to carry one person sitting on the seat. Before using your Handicare 1100 stairlift please read this manual thoroughly to familiarise yourself with how to operate your stairlift. Your stairlift has been constructed with state of the art technology and in accordance with recognised safety regulations so you can be sure that you have a safe and reliable solution.

Nevertheless, incorrect use of the stairlift may cause risk to the user or third parties, damage to your stairlift or other goods. Use your stairlift only when it is in good working order, in accordance with the intended use described above and with due attention to this manual. Always keep this manual near the stairlift.

Users who cannot see and hear should receive assistance when using the stairlift.

TECHNICAL SUPPORT

Should issues arise that are not covered by this document please contact your supplier.

Their Service department staff will always be pleased to help you. To help them identify your product please have the following details to hand when you call them:

- this manual
- your name, address, postcode, town and telephone number stairlift serial number which can be found on the label located on the power pack underneath the seat.

SAFETY INSTRUCTIONS

When using the stairlift always observe appropriate safety regulations regarding risk of electric shock and physical injury.

1. Provide an unobstructed environment by ensuring that the entire track is free of obstacles. Keep the rail free from oil and ice.
2. Take environmental influences into account. Do not expose the lift to damp or extreme sunlight. Make sure that the environment is well lit.
3. The stairlift should only be used by individuals who have received the appropriate guidance. On installation the supplier is required to demonstrate how to use this stairlift and the user is required to sign the declaration confirming that this instruction has been received.
4. Stairlifts should only be used for carrying one person: do not use your stairlift to carry pets, goods or more than one person at a time, even if the load is below the maximum user weight. Keep children and pets away from the stairlift.
5. Remove the key after use to prevent unauthorised use.
6. Keep limbs and clothing away from moving parts. Make sure that wide or loose clothing cannot become trapped between the chair and the stairs or rail.
7. Ensure a safe position during use. Always ensure a correct, stable position. Please sit as far back in the seat as you can and place your feet as close to the centre of the footplate as you are able to. Always use the safety belt and never stand on the stairlift.
8. Take care at all times. Concentrate on the operation of the stairlift as releasing the operating toggle will cause the lift to stop. To restart the lift wait 3 seconds and push and hold the toggle in the direction of travel desired until you reach the landing or hallway.
9. Check for damage. Before use check the lift for external damage. Don't use the lift if parts are defective or missing (e.g. track end caps of the slidetrack). Report unusual noises, such as squeaking and creaking, to your supplier. To avoid accidents repairs and preventive maintenance should only be carried out by engineers qualified to do so.
10. The stairlift must not be used in the event of a fire or as a means of escape during a fire.
11. If the stairlift is not fitted with a communication device, we recommend that you always have one to hand.

Structure


The strength and the suitability in general of wall, staircases, terraces, upper floors, banisters, plinths etc. which are to support or surround the installation have been verified and ensured by the customer, who therefore accepts all relative responsibilities. Neither Handicare nor its authorised dealers may be held responsible for the suitability of the structures which surround and support the installation.

MAINTENANCE

Cleaning

Clean the stairlift regularly with a damp cloth; avoid excessive use of water. Clean the rail regularly with a vacuum cleaner with a soft brush. Ensure the track is kept free of dirt and dust. Any dirt on the rail can easily be removed with a slightly damp cloth. Under no circumstances should you use cleaning product when cleaning the rail.

IMPORTANT: FRICTION DRIVE - DO NOT APPLY LUBRICANT/GREASE TO RAIL After a while, the wheels may leave marks on the rails.

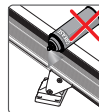Preventive maintenance

In addition to cleaning your stairlift, periodic preventive maintenance is necessary for continued long-term use. Your stairlift must be serviced at least once a year by a qualified person. If your stairlift has more than two users we recommend a minimum of two services per year. Please make an electrical socket available to provide local lighting if required during inspection and servicing. Lighting to a minimum value of 50 lux shall be provided at the boarding points whilst the stairlift is in use. Most light bulb packaging will detail the lux value of the light bulb.

GUARANTEE AND LIABILITY

Term of warranty – this warranty is valid for a period of 24 months from the date that the product ships from our factory. The purchase price for the product must be paid in full in order for the manufacturer to release parts under this warranty.

Coverage – this warranty applies to the repair or replacement, at the manufacturer's discretion, of parts that fail due to defective material or workmanship. The manufacturer may, at its discretion, provide factory reconditioned parts. This warranty is provided to the Authorised Handicare Dealer on behalf of the final purchaser of the product and is not transferable. The manufacturer's warranty does not cover labour charges for the removal, repair or replacement of warranty parts.

Damage caused by normal wear and tear, overloading or misuse and the replacement of batteries in the remote control is not covered by this guarantee. We recommend that your stairlift is serviced annually by an appropriately qualified stairlift engineer.

Under no circumstances can the manufacturer accept responsibility for death, personal injury, damage to property, or incidental, unforeseen consequences and/or costs or damage resulting from the use of your stairlift.

The carrying out of alterations to the product is not permitted and doing so will invalidate the warranty. The manufacturer is not liable for damage resulting from such alterations.

The manufacturer accepts no liability for damage or injury caused by the non or inadequate observance of the safety regulations and instructions in this manual, or by negligence during the installation, use, maintenance and repair of this stairlift.

Installation and servicing should only be carried out by dealers or installers/service engineers authorised by the manufacturer in accordance with local regulations.

The wall mounted remote controls operate on a infrared frequency; Handicare is unable to accept responsibility for system inoperability in the unlikely event of interference from signals external to this stairlift.

Technical specifications

Supply voltage 100-240 V~
 Frequency 50/60 Hz
 Current 1.0 A
 Power consumption 0-36 watt
 Fuses 20 A
 Max.load (inclination 25°-45°) 140 kg
 Max.load (inclination 45°-50°) 130 kg
 Max.load (SlideTrack) 120 kg

Speed max. 0.15 m/s
 Ambient temperature 0 - 40 °C
 Relative humidity 20 - 80 %
 Usage 10 rides per day
 Noise 56 dB
 Use Indoor

Technical specifications NA

Supply voltage 120 V~
 Frequency 60 Hz
 Current 0.83 A
 Power 20 watt
 Fuses 20 A
 Max.load (inclination 25°-45°) 308 lbs
 Max.load (inclination 45°-50°) 286 lbs
 Max.load (SlideTrack) 264 lbs

Speed max. 0.49 f/s
 Ambient temperature 32 - 104 °F
 Relative humidity 20 - 80 %
 Usage 10 rides per day
 Noise 56 dB
 Use Indoor

If your lift does not have a "two-way voice communication device" you can order this option at any time, which can be connected to the stairlift if necessary, if the personal (living) circumstances of the user have changed in such a way that the connection of such a device to the stairlift is an appropriate solution for the possible risk that the user cannot call for help quickly enough when he / she gets stuck on the stairlift due to a malfunction or otherwise.

The information provided in this manual is based on general specifications regarding the construction, material properties and working methods known to us at the time of publication; alterations and improvements are therefore anticipated.
The Dutch manual is the original language.

DECLARATION ENGINEER	
I/We certify that onI/we thoroughly examined this lift and that the foregoing is a correct report of the result. I/We declare that the erected stairlift is compound with approved components and installed according to the instructions in the installation manual. I/We also declare that I/we have instructed the user by telling him/her all users practice. The user is independently able to use the lift safely without further instruction. The user manual has been handed over to the user.	
Name Engineer:	Name Engineer:
Qualifications:	Qualifications:
Signing date:	Signing date:
Signature:	Signature:
Address:	If employed by a company/association>name/address:

DECLARATION INSTRUCTED PERSON	
Herewith confirms the user of the stairlift that the above mentioned engineer(s) has declared all users practice and that he/she is known with the complete operation as instructed person.	
Name:	Adress:
City:	Phone number:
Signing date:	Signature:

EC DECLARATION OF CONFORMITY	
We, Handicare Newtonstraat 35, 1704 SB Heerhugowaard, the Netherlands, declare that the Handicare 1100 is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.	
The stairlift has been designed and manufactured in accordance with the following standards: EN 81-40, ISO 9386-2, EN ISO 12100-1, EN ISO 12100-2, EN ISO 14121, EN60204-1	
Liftinstituut B.V., Bukslotermeerplein 361, 1025 XE Amsterdam, The Netherlands. Identificationnr 0400 has issued an EC-certificate in compliance with machinery directive 2006/42/EC with reference NL17-400-1001-015-10.	
The Technical Construction File for the above Stairlift is retained at the manufacturers address. A copy is retained at the Liftinstituut B.V. office.	
Heerhugowaard, The Netherlands, 15 april 2023.	Serial no. unit:
J. Hopman, Managing director	Project:
Date put into service:	Date:

The declaration of conformity is legal after the installation checklist and the declaration instructed person have been signed off by the engineer and the user.
 The installation checklist and the declaration instructed person shall be saved by the dealer and made available to Handicare on request.

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EC DECLARATION OF CONFORMITY

We, Handicare Newtonstraat 35, 1704 SB Heerhugowaard, the Netherlands, declare that the Handicare 1100 is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.

The stairlift has been designed and manufactured in accordance with the following standards: EN 81-40, ISO 9386-2, EN ISO 12100-1, EN ISO 12100-2, EN ISO 14121, EN60204-1

Liftnstituut B.V., Buikslotermeerplein 361, 1025 XE Amsterdam, The Netherlands. Identificationnr 0400 has issued an EC-certificate in compliance with machinery directive 2006/42/EC with reference NL17-400-1001-015-10.

The Technical Construction File for the above Stairlift is retained at the manufacturers address. A copy is retained at the Liftnstituut B.V. office.

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